



Comfortable
Support
after
Breast Cancer
Surgery



JOBST® Bellisse®
Medical Compression Garment

JOBST® Bellisse®



Padded straps are adjustable.

- Comfortable wearing without shoulder pressure

Special Compression Comfort® inner support system provides circumferential compression.

- Comfort, day and night

The natural stretch cups have an inner pocket to accommodate the lateral pad or the chest-wall pad.

- Compression promotes lymph drainage

Secure front zipper with hooks and eyes to hold garment in place during zipping.

- For easy donning and doffing

Soft, non-rolling lower band keeps garment from riding or rolling up.

- Soft on the skin
- Smooth while sleeping
- Unobtrusive under clothing


Comfort that makes a difference following breast cancer surgery

An innovative approach to breast and chest wall edema following breast cancer surgery

JOBST® Bellisse® with its special **Compressure Comfort®*** inner support system is the innovative solution for the management of breast and chest wall edema. **JOBST® Bellisse®** is designed to meet the specific needs of women suffering from lymphedema, post-surgical discomfort, post-radiation edema and/or fibrosis. This medical compression garment can be worn day and night, is fully adjustable, and provides gentle compression with and without the addition of breast pads. It helps to reduce discomfort and provides gentle, even compression around the torso, facilitating lymph drainage from the breast and chest wall.

JOBST® Bellisse® was developed by Lesli R. Bell, DPT, CLT-LANA, a specialized lymphedema therapist. Her interest in women's health and oncology led her to pursue an idea for a compression garment for the management of lymphedema of the chest wall. **JOBST® Bellisse®*** is the first compression product made especially for the needs of the post breast cancer surgery patient.

Highly adjustable with three position back closure and with extender providing additional 3 rows of eye loops.

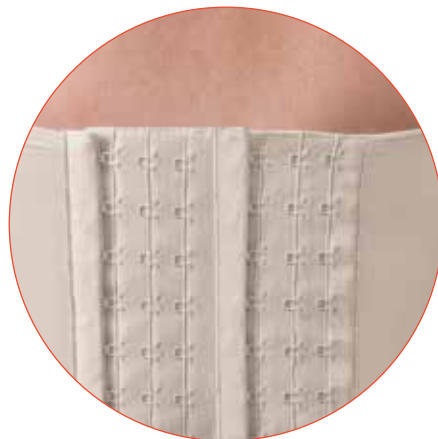

Colors



Buff



Black


Measuring Guidelines

Step 1: Determine your client's Band Size:

- Measure circumferentially below the bust where a regular bra rides.
- Drop the tape 2"-3" and measure around the bottom of the rib cage.
- Use the larger of the two rib cage measurements for your client's band size. Round odd numbers up.

Step 2: Determine your client's Cup Size:

- Measure circumferentially around the fullest part of the bust.
- Subtract largest ribcage measurement from the bust measurement to determine cup size.

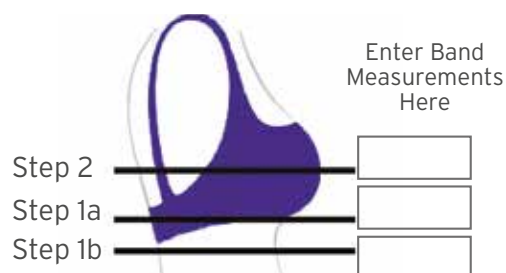

Enter Band Measurements Here

Difference	Cup Size
2" or less	A/B
2"-4"	C/D
4"-6"	DD/E

If more than 6" go up one band size.

Bellisse Size:

*Bellisse® and Compressure Comfort® are registered trademarks of Lightning2, LLC.

JOBST® JOVIPADS

Available in different shapes, the **JOBST® JoViPads** address particular areas of the breast and chest wall affected by pitting edema and/or fibrosis. The pads provide gentle and prolonged pressure to soften congested tissue and encourage evacuation of lymph fluid to functional pathways. Designed for use under the **Bellisse®** Medical Compression Garment.



Lateral Pad



Mini Axilla Pad



Chest-wall Pocket Pad



Post-Lumpectomy Pad



Serratus Anterior Pad



Unilateral Post-Mastectomy Pad



Double Mastectomy Pad



Axilla Pad



Inframammary Pad



Cleavage Pad



Padded Insert

Bellisse® exclusively distributed by JOBST®

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,
an Essity brand

BSN Medical Inc., an Essity company
5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325
To order toll-free: Tel. (+1) 800 537 1063

62755 R2 ©2023 Essity HMS North America Inc. G23