

OBST// Compression therapy made easier!







. . . .

Scan the QR code



The donning aid for closed-toe medical compression stockings and tights with a unique magnetic system

Before you start:



Important! Don't push the foot to the very end of the Magnide®, leave some free space at the end



In case the compression garment has a silicon border, make sure to turn it outwards

Donning instructions




















Preparation for re-use

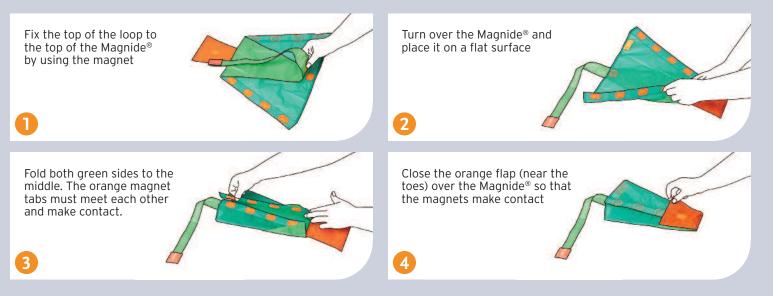

Arion donning and doffing aids for medical compression garments

Easy to Use

- Minimize friction between compression garment and skin
- · Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

Safe to Apply

- May help reduce the risk of skin damage by minimizing friction
- · May contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help avoiding garments from overstretching and distortion

Simple Solutions

- · Offer application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

SIZES Arion Magnide®

Size	Shoe Woman	size Man	BNR
Medium	≤7.5	≤6	7957100
Large	6 - 9.5	6.5 - 10	7957101
X-Large		8.5	7957102
XL also suitable for extra wide feet (circumference up to 11"/28 cm)			

Should you need another size, Arion can customize the product according to your needs.

Please contact your JOBST[®] sales representative for more information. JOBST[®] is the preferred distributor of Arion Slide Solutions in the USA and Canada.



BSN Medical Inc., an Essity company 5825 Carnegie Blvd., Charlotte, NC 28209-4633 Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325 To order toll-free: Tel. (+1) 800 537 1063 www.jobst-usa.com

Essity 1275 North Service Road West, Suite 800 Oakville, ON Canada L6M 3G4 Tel. 1-877-978-5526 Fax 1-877-978-9703 www.jobstcanada.com Manufactured by:

A RION Rutherford 11 . 6422 RE Heerlen The Netherlands www.slide-solutions.com

64190 RN ©2021 BSN Medical Inc. B21