




Scan the QR code to watch the video instructions

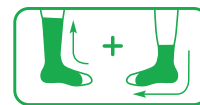


On & Off

Arion Sim-Slide®

The optimal multifunctional application aid for **donning and doffing** medical compression stockings and tights with a closed toe


Before you start:

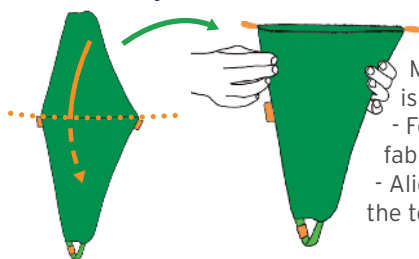

Make sure the product is ready for use:
 - Fold back the extra fabric into the „green bag“
 - Align the fold line with the top of the orange labels








Important! Don't push the foot to the very end of the Arion Sim-Slide®, leave 2 or 3 fingers space in front of the toes.








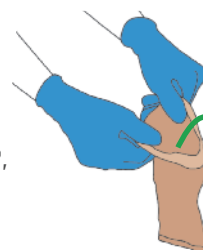


In case the compression garment has a silicon border, make sure to turn it outwards

Donning instructions

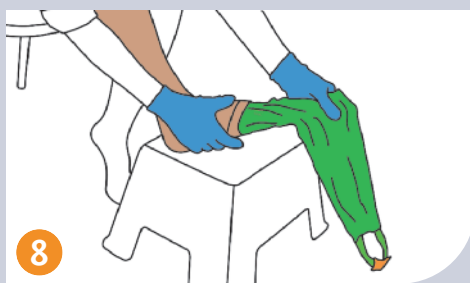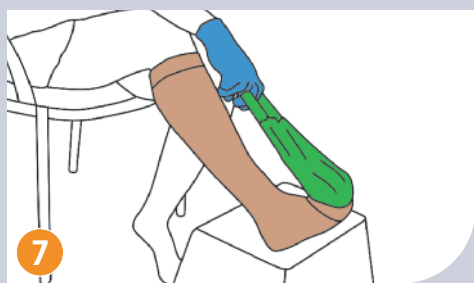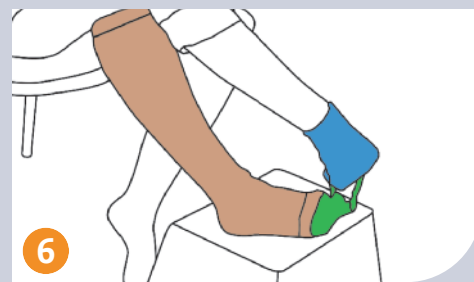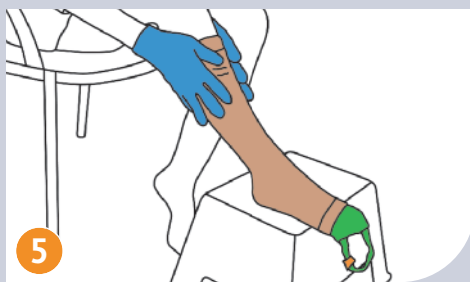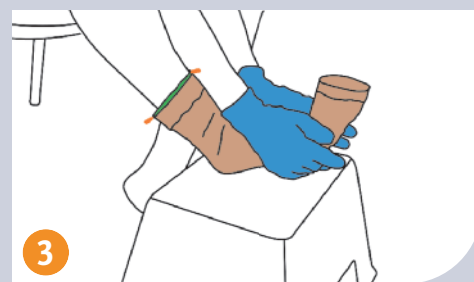

Doffing instructions

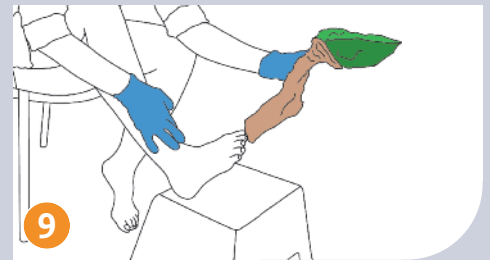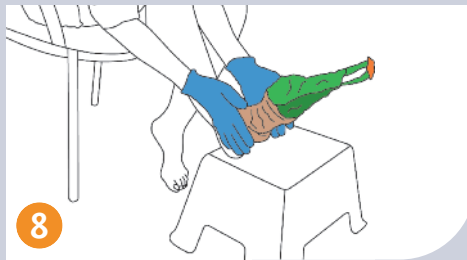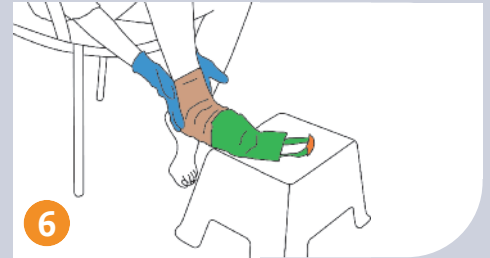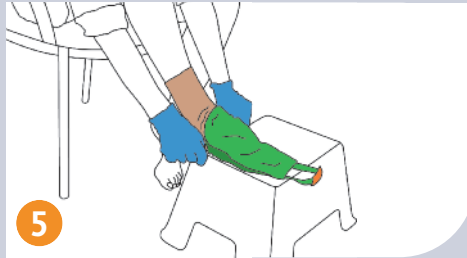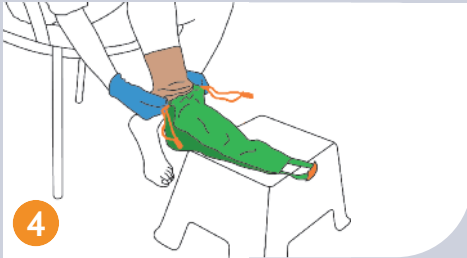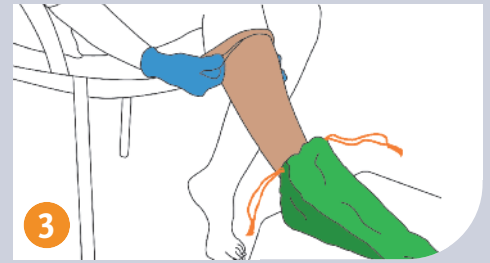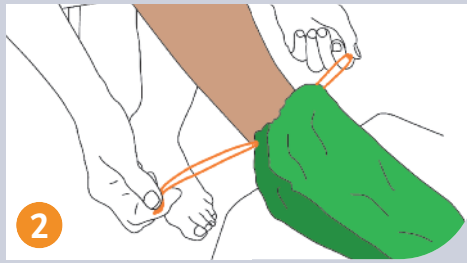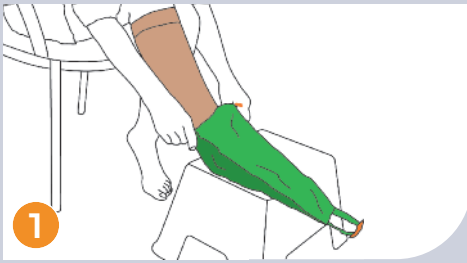

Arion donning and doffing aids for medical compression garments

Easy to Use

- Minimize friction between compression garment and skin
- Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

Safe to Apply

- Reduce risk of skin damage ; can be especially beneficial in case of high skin sensitivity or leg ulcers
- Contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help maintaining the original compression longer and avoiding garments from overstretching and distortion

Simple Solutions

- Offer application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

SIZES Arion Sim-Slide®

Size	Shoe size		BNR
	Woman	Man	
Small	≤ 4 - 6	3 - 4.5	7063301
Medium	6.5 - 8	5 - 6.5	7063302
Large	8.5 - 9.5	7 - 8	7063303
X-Large	≥ 10	≥ 8.5	7063304

Should you need another size, Arion can customize the product according to your needs.

Please contact your **JOBST®** sales representative for more information.
JOBST® is the preferred distributor of Arion Slide Solutions in the USA and Canada.



Manufactured by:

ARION

Rutherford 11 . 6422 RE Heerlen
 The Netherlands
www.slide-solutions.com



BSN Medical Inc., an Essity company
 5825 Carnegie Blvd., Charlotte, NC 28209-4633
 Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325
 To order toll-free: Tel. (+1) 800 537 1063
www.jobst-usa.com

Essity
 1275 North Service Road West, Suite 800
 Oakville, ON Canada L6M 3G4
 Tel. 1-877-978-5526 Fax 1-877-978-9703
www.jobstcanada.com