



Scan the QR code to watch the video instructions



## Arion Easy-Slide

The original **donning aid** for medical compression stockings and tights with an open toe.



On



Scan the QR code to watch the video instructions





## Arion Easy-Slide Arm

The user friendly **donning aid** for medical compression arm sleeves








On

### Donning instructions

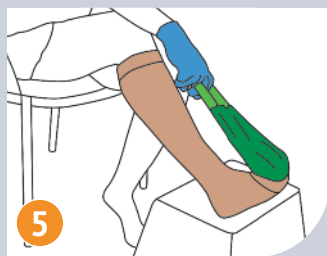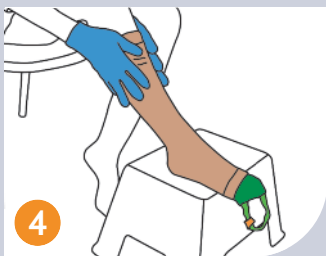


### Donning instructions

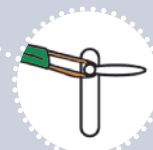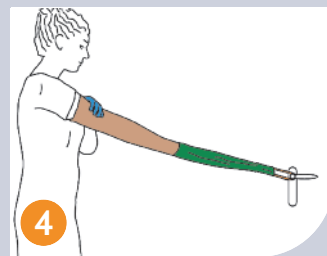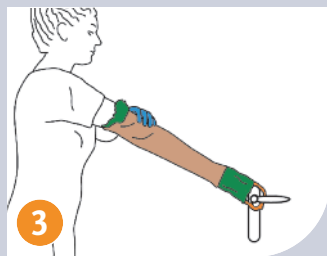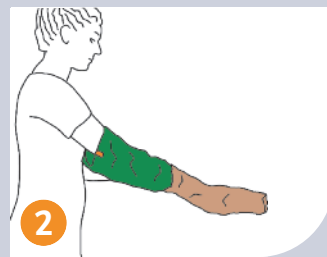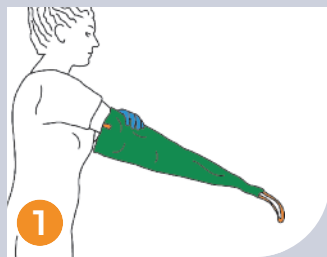


Use a door handle to make it easier pulling out the Easy-Slide Arm

### Before you start:

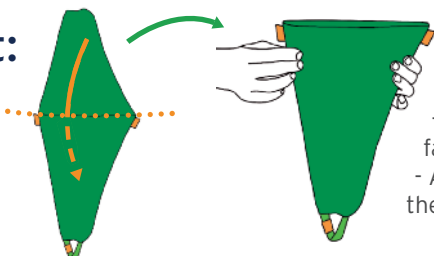

Make sure the product is ready for use:  
 - Fold back the extra fabric into the „green bag“  
 - Align the fold line with the top of the orange labels

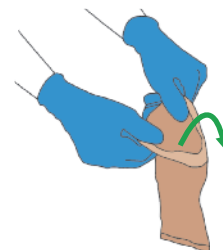

In case the compression garment has a silicon border, make sure to turn it outwards

## Arion Easy-Slide



## Arion Easy-Slide Arm



### Arion donning and doffing aids for medical compression garments

#### Easy to Use

- Minimize friction between compression garment and skin
- Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

#### Safe to Apply

- Reduce risk of skin damage ; can be especially beneficial in case of high skin sensitivity or leg ulcers
- Contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help maintaining the original compression longer and avoiding garments from overstretching and distortion

#### Simple Solution

- Offer application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

### SIZES Arion Easy-Slide

Size	Shoe size		BNR
	Woman	Man	
X-Small	≤ 3.5	≤ 2.5	7965803
Small	4 - 6	3 - 4.5	7965804
Medium	6.5 - 8	5 - 6.5	7965802
Large	8.5 - 9.5	7 - 8	7965902
X-Large	≥ 10	≥ 8.5	7966001

### SIZES Arion Easy-Slide Arm

Size	Arm length*	BNR
Medium	21.5" (55.5cm)	7966102
Large	24.5" (62.5cm)	7510001

\*Arm length from the tip of the middle finger to the armpit

Should you need another size, Arion can customize the product according to your needs.

Please contact your **JOBST®** sales representative for more information. **JOBST®** is the preferred distributor of Arion Slide Solutions in the USA and Canada.



BSN Medical Inc., an Essity company  
5825 Carnegie Blvd., Charlotte, NC 28209-4633  
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325  
To order toll-free: Tel. (+1) 800 537 1063  
www.jobst-usa.com

64171 RN ©2021 BSN Medical Inc. B21

Essity  
1275 North Service Road West, Suite 800  
Oakville, ON Canada L6M 3G4  
Tel. 1-877-978-5526 Fax 1-877-978-9703  
www.jobstcanada.com

Manufactured by:

**ARION**

Rutherford 11 . 6422 RE Heerlen  
The Netherlands  
www.slide-solutions.com