

JOBST® FarrowWrap®

## WHY JOBST® FARROWWRAP®...

#### HOW IT WORKS

How does a JOBST® FarrowWrap® achieve compression?

- Utilizing special fabrics with smart elastic shortstretch bands comprised of multi-layered fabrics
- Optimal width of bands and amount of band overlap from bottom to top


It is important that every band of a JOBST® FarrowWrap® garment be applied at or near end-stretch. Having a consistent amount of tension throughout the garment helps to ensure graduated compression.

The Elastic Short-Stretch<sup>™</sup> fabrics provide sustained resting compression while also augmenting the calfmuscle pump to deliver high working compression.

#### **BENEFITS**


- Easy use of the simple VELCRO® Brand hook & loop system saves therapists' and patients' time, and supports easy donning and doffing for maximum or optimal patient independence
- Since the product is adjustable, fluctuating edema levels can be more easily managed by the patients themselves, empowering them to control their own care
- The variety of styles and sizes available in Ready-to-Wear (RTW), Trim-to-Fit (TTF), Custom-Made (CM) ensures there is a FarrowWrap® to meet a broad range of clinical need



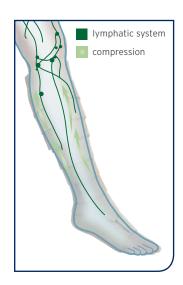


## CHARACTERISTICS OF SHORT-STRETCH TECHNOLOGY

- Allows creation of products with different compression levels at end-stretch
- Provides augmentation of the body's natural muscle pump action
- Maximizes calf muscle function

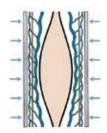

The pressure, exerted onto the body from the outside when the muscles at rest, is called "resting pressure"

- Helps minimize swelling throughout the day once garment is donned
- Provides higher compression when the patients stand or are active and less when they are not

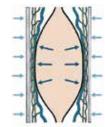

The pressure exerted onto the body from the inside as the muscle activates and expands, is called the "working pressure"

Short-stretch bandages deliver a low resting pressure and a high working pressure, producing a massaging effect on the calf muscle when the patient is active, which in turn produces a subsequent increase in lymphatic return. This makes short-stretch systems very effective in the management of lymphedema.

## THE JOBST® FARROWWRAP® PRODUCT RANGE

Adaptable and Affordable, Comfortable and Containing

## **Product Range**

#### LITE

A double-laminated fabric suitable for mild to moderate compression needs, making it especially suitable for home care use / self-management patients. The fabric is both durable and breathable.

#### Available versions

- Upper and lower limb
- Trim-To-Fit (TTF)
- Ready-to-Wear (RTW)
- Custom-Made (CM)

#### Mild to moderate chronic edema / lymphedema Armpiece Armpiece Ambidextrous Footpiece Legpiece Hand Gauntlet Foam Squares (Ready-to-Wear only)

#### **STRONG**

Kneepiece

Premium fabric with strong compression and soft inner layer suitable for patients with a good limb shape and moderate to severe edema.

#### Available versions

Upper and lower limb

Thighpiece

- Trim-To-Fit (TTF)
- Ready-to-Wear (RTW)
- Custom-Made (CM)

Moderate to severe chronic edema / lymphedema with a good limb shape











Footpiece Leapiece

Kneepiece

Thighpiece

#### **CLASSIC**

The original material that has been clinically proven over many years of use on thousands of patients worldwide.

#### Available versions

- Upper and lower limb
- Custom-Made (CM)
- Ready-to-Wear (RTW)
  - Moderate to severe chronic edema / lymphedema







#### MEDICARE APPROVED\*\*\*

#### **BASIC**

A Value fabric with quality compression. Uses double-sided VELCRO® Brand product so it is possible to change band length and trim as limb reduces.

#### Available versions

Lower limb

Ready-to-Wear (RTW)

30-40 mmHg\* Moderate to severe chronic edema / lymphedema





<sup>†</sup>This sock is a liner and only provides compression at the foot.

## 4000

The inner sleeve and 4-band design make for easy application. Anatomically contoured to prevent gapping.

#### Available versions

Lower limb

• Ready-to-Wear (RTW)

30-40 mmHg\* Moderate to severe chronic edema / lymphedema





<sup>†</sup>This sock is a liner and only provides compression at the foot



# PRODUCT ASSORTMENT

	Footpiece	Legpiece	Foot Compression Liner	Thighpiece	Kneepiece	Armpiece	Ambidextrous Hand Gauntlet Foam Squares
Mild to Moderate 20-30 mmHg		2233					
JOBST <sup>®</sup> FarrowWrap <sup>®</sup> LITE	RTW, CM, TTF	RTW, CM, TTF		RTW, CM	RTW	RTW, CM, TTF	RTW
JOBST® FarrowHybrid® ADI			RTW				
Moderate to Strong 30-40 mmHg		2233		***************************************			
JOBST <sup>®</sup> FarrowWrap <sup>®</sup> STRONG	RTW, CM, TTF	RTW, CM, TTF		RTW, CM	RTW	СМ	
JOBST® FarrowWrap® CLASSIC	RTW, CM	RTW, CM		RTW, CM		СМ	
JOBST® FarrowWrap® 4000		RTW					
JOBST® FarrowHybrid® ADII			RTW				
JOBST® FarrowWrap® BASIC	RTW	RTW					

RTW: Ready-To-Wear • CM: Custom-Made • TTF: Trim-To-Fit

	Hand Gauntlet Foam Squares	Legpiece	Arm- or Legpiece	Toe Cap	Glove
Accessories					
JOBST® FarrowWrap® LITE	RTW				
JOBST® Farrow GarmentGrip™ Roll*		RTW	RTW		
JOBST® FarrowWrap® Toe Cap" and Gloves				RTW	RTW

**JOBST® FarrowWrap® GarmentGrip** can be cut to size to fit the shape and contour of the body and VELCRO® brand hook-and-loop fasteners can be added to secure in place. For triangular-shaped legs, it is recommended that it be worn over the legpiece to secure the wrap system in place.

**JOBST® FarrowBand** is washable and reusable. It is available for the upper and lower limb in differing roll lengths and extra wide widths, and can also be used to manage mid-line edema.

\*\*JOBST® FarrowWrap® Toe Caps and Gloves are available in 15-20 and 20-30 mmHg.

## Pad & Shape

JOBST® FarrowFoam comes in a variety of sizes and rolls or sheets. It is great for shaping any irregular limb shapes or providing padding to sensitive areas.



## Ready-To-Wear (RTW)

Garments are pre-made pieces to fit the majority of patient sizes.

#### Trim-To-Fit (TTF)

Garments offer one garment for both active/reduction and maintenance therapy. A Trim-To-Fit garment can be trimmed to adapt as extremity reduces. It can fit most patients in a single size.

#### Custom-Made (CM)

Garments are manufactured to the specific limb measurements of each patient to provide a better fit for those limb shapes that do not fit within the RTW size range.

## UPPER EXTREMITY

#### JOBST® FarrowWrap® ARMPIECE - SUITABLE FOR 24-HOUR WEAR

#### LITE RTW Armpiece\*



LITE RTW armpiece is an anatomically contoured, effective solution to control upper extremity edema. It is easy to apply and remove with one hand and designed for mild to moderate edema control.

• 20-30 mmHg

#### LITE TTF Armpiece\*



The LITE TTF armpiece can be customized at the point of purchase and will fit most limb sizes. The LITE TTF armpiece can be retrimmed as the patient's edema reduces.

• 20-30 mmHg

#### LITE Custom Armpiece\*



The LITE custom armpiece consists of overlapping bands of short-stretch fabric. It generally requires two hands to apply. The short-stretch bands allow garment to be applied at end stretch.

• 20-30 mmHg

#### CLASSIC Custom Armpiece\* STRONG Custom Armpiece\*



CLASSIC custom armpiece is for more severe edema requiring high levels of compression. It is only available as a custom garment and generally requires two hands to apply. Patients with very large arms or moderate to severe swelling would benefit from a CLASSIC garment.

· 30-40 mmHg



The STRONG custom armpiece is used in the same situations as CLASSIC but has the added benefit of ease of maintenance.

• 30-40 mmHg

<sup>\*</sup> Colors: Tan, Black; Sizes: S, M, L; Available in Left or Right, and Short, Regular & Long Lengths

#### JOBST® FarrowWrap® HANDPIECE - SUITABLE FOR 24-HOUR WEAR



#### JOBST® FarrowWrap® LITE Hand Gauntlet 20-30mmHg\*

JOBST® FarrowWrap® LITE Hand Gauntlet is ambidextrous and comes with 4mm and 8mm foam squares for added compression to the dorsal hand. Suitable for mild to moderate edema.

#### Benefits that give your patient greater freedom to enjoy life:

- Foam inserts help reduce edema that can pool on the top of the hand
- · Ambidextrous, i.e. can be used on either hand
- Durable and comfortable

#### JOBST® FarrowWrap® GLOVE



#### JOBST® FarrowWrap® Glove 15-20mmHg\* and 20-30mmHg\*

JOBST® FarrowWrap® Glove is durable, latex-free, and provides gentle compression for patients with mild to moderate edema. Its ultra-thin fabric and fine seams ensure great wearing comfort.

#### Benefits that give your patient greater freedom to enjoy life:

- Available in eight sizes, covering a broad range of patients
- Smooth compression fabric is made from a blend of soft yarns
- Fingers can be easily trimmed with scissors

#### LINER



#### Flesh Tone Silver Liner (Antimicrobial)\*\*

Liner is only required for trim-to-fit custom armpiece. There is no compression in liner.

The liner provides a smooth base and skin protection prior to applying the outer wrap piece. In addition, daily washing of the outer wrap piece is not recommended due to the nature of the material and the VELCRO® brand hook-and-loop fasteners. However, a fresh liner can be worn daily as the liner can be washed as frequently as needed.

<sup>\*</sup> Colors: Tan; Sizes: XS, S, M, L

<sup>\*\*</sup> Colors: Tan; Sizes: S, L

## LOWER EXTREMITY

JOBST® FarrowWrap® THIGHPIECE AND LEGPIECE - SUITABLE FOR 24-HOUR WEAR

LITE Leapiece\*\*



CLASSIC Legpiece\*\*



STRONG Legpiece\*\*



**BASIC Legpiece\*\*** 



LITE Thighpiece





**CLASSIC Thighpiece STRONG Thighpiece** 

JOBST® FarrowWrap® LITE\* is designed for mild to moderate edema control. Available in Ready-To-Wear (RTW), Trim-To-Fit (TTF), and Custom-Made (CM). LITE garments can be machine washed and dried.<sup>†</sup>

· 20-30 mmHg

JOBST® FarrowWrap® CLASSIC\* is designed for moderate to severe edema control. It is available in Ready-To-Wear (RTW) and Custom-Made (CM) garments. Hand wash only.<sup>†</sup>

• 30-40 mmHg

JOBST<sup>®</sup> FarrowWrap<sup>®</sup> STRONG\* is designed for moderate to severe edema control and is available in Ready-To-Wear (RTW) and Custom-Made (CM) sizes. STRONG garments can be machine washed and dried.<sup>†</sup>

• 30-40 mmHg

JOBST® FarrowWrap® BASIC\* is an exceptional value and features trimmable bands and adjustable VELCRO® Brand products so the garment can be trimmed as the limb reduces. BASIC garments can be machine washed and dried.<sup>+</sup>

• 30-40 mmHg Available in Legpiece only.

<sup>\*</sup> Colors: Tan; Sizes: XS, S, M, L, XL; Available in Short, Regular and Tall Lengths

Colors: Tan; Sizes: XS, S, M, L, XL; Available in Regular and Tall Lengths

<sup>&</sup>lt;sup>†</sup> Follow care instructions in the product packaging.

### JOBST® FarrowWrap® LEGPIECE - SUITABLE FOR 24-HOUR WEAR



#### JOBST® FarrowWrap® 4000

JOBST® FarrowWrap® 4000 legpiece is designed for moderate to severe edema and is easy to apply with just four bands and inner sleeve. It is available in RTW sizing. 4000 garments can be machine washed and dried.



#### JOBST® FarrowWrap® Kneepiece

JOBST® FarrowWrap® Kneepiece is a RTW garment available in LITE and STRONG. LITE can be used in combination with a JOBST® FarrowWrap® LITE Thighpiece, whereas STRONG complements the JOBST® FarrowWrap® STRONG and CLASSIC Thighpieces.



# Flesh Tone Silver Liner\*\* (Antimicrobial)

• Fits up to 70cm calf circumference

The liner provides a smooth base and skin protection prior to applying the outer piece, and can be washed daily.

#### Customizing garments made easy with Trim-to-Fit

TTF garments can be customized on site to fit a variety of patients. You can easily retrim the garment if the patient's edema reduces. There are two TTF sizes which cover 6 standard sizes. TTF is available in STRONG and LITE fabrics.

On triangular shaped legs we recommend  $\mathbf{JOBST}^{\circ}$  Farrow  $\mathbf{GarmentGrip^{TM}}$  be worn over our legpiece to help hold the garment in place.

<sup>\*</sup> Colors: Tan, Black; Sizes: XS, S, M, L

<sup>\*\*</sup> Colors: Tan; Sizes: S, L

## LOWER EXTREMITY

## JOBST® FarrowWrap® FOOTPIECE - SUITABLE FOR 24-HOUR WEAR

#### LITE Footpiece\*



LITE footpiece provides compression for mild to moderate edema.

· 20-30 mmHg

#### **STRONG Footpiece\***



STRONG footpiece provides compression for moderate to severe edema.

• 30-40 mmHg

#### **CLASSIC Footpiece\***



CLASSIC footpiece is clinically proven in managing edema. Durable construction.

• 30-40 mmHg

#### **BASIC Footpiece\***



BASIC footpieces provide compression for moderate to severe edema.

• 30-40 mmHg

#### FarrowWrap® Toe Cap



Enjoy the unique benefits of the JOBST® FarrowWrap® Ready-to-Wear Toe Cap.

The durable, latex-free Toe Caps can be used on either foot and resist rolling and fraying when trimmed.

- 15-20 and 20-30 mmHg
- Tan and Black colors

#### **Customizing garments**

TTF footpieces can be instantly customised using our proprietary TTF innovation. Must be trimmed before donning.

 $<sup>^{*}</sup>$  Colors: Tan; Sizes: XS, S, M, L, XL; Available in Regular and Long Lengths

# THE THERAPEUTIC CONTINUUM OF CARE FOR EVERY PATIENT JOURNEY

It is critical to take proactive measures using an integrated approach combining compression therapy with treatments that support optimal wound healing. Essity supports the Continuum of Care approach that enhances outcomes by combining therapies and choosing patient-centered devices to optimize results.





JOBST® is the #1 Physician recommended brand of compression hosiery in the U.S.

To learn more about

JOBST® products and services, call

1-800-537-1063

or visit

https://order.bsnmedical.com/us

\* The average compression for a medium sized ankle. VELCRO® is a Registered Trademark of Velcro BVBA. Please refer to the product label and / or package insert for full instructions on the safe use of these products.

JOBST°,

#### \*\*\*REIMBURSEMENT DISCLAIMER:

The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA as the Pricing, Data Analysis, and Coding (PDAC) Contractor. Their website, located at https://www.dmepdac.com, maintains a searchable database for HCPCS codes. Essity, however, makes no representations as to the accuracy of the information contained within the PDAC database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine and submit appropriate codes, charges, and modifiers for services rendered.





