

# JOBST® JOVIPAK® HEAD AND NECK GARMENTS

JOBST® JoViPak® Head and Neck Garments are for lymphedema patients who want a more **comfortable** and **easier** alternative to bandaging. These handmade compression garments aim to support lymph fluid drainage because of their continuous flow channeling and blended density chip foam construction.

## STANDARD CHIN STRAP



Designed to address edema of the neck, mandible, and the anterior/lateral aspects of the face (partially covers the ears). The open neck seam accommodates your choice of padding. The VELCRO® Brand closures at the neck and top of head (or optional Occipital Head Strap) can be adjusted to patient's comfort level, making the garment suitable for overnight wear. Should be ordered with the Universal Neck Pad.

## EXTENDED CHIN STRAP



Designed to address edema of the neck, mandible, and the lateral aspects of the face; covers the ear, the lateral aspect of the head and behind the ear. The open neck seam accommodates your choice of padding. The VELCRO® Brand closures at the neck and top of head (or optional Occipital Head Strap) can be adjusted to patient's comfort level, making the garment suitable for over-night wear. Should be ordered with the Peri-Auricular Neck Pad.

## FULL FACE MASK



Designed to cover the entire face except the nose and lips. This mask has adjustable VELCRO® Brand closures on the top and back of the head (the Occipital Head Strap), the neck, and above the top lip. The adjustable strap across the upper half of the face allows for full vision when needed.

## UNIVERSAL NECK PAD



Designed to address bilateral head and neck edema with swelling of the neck in front of or behind the ear. If swelling is present in front of the ear, order this pad with a Standard Chin Strap. If swelling is present behind the ear, order this pad with an Extended Chin Strap.

## PERI-AURICULAR NECK PAD



Designed to address bilateral head and neck edema with swelling of the neck in front of or behind the ear. Order this pad with an Extended Chin Strap.

## HALF FACE MASK




A modified version of the Chin Strap, this loosely foam-filled garment covers the cheek, eye, and forehead. The VELCRO® Brand closures at the neck, top, and back of head (Occipital Head Strap) can be adjusted to patient's comfort level, making the garment suitable for overnight wear.

## EYE PAD



Designed to provide bilateral padding over the upper half on the orbital area face. If used under the JOBST® JoViPak® Full Face Mask for therapeutic benefit, it may be ordered without adjustable head strap.

# HEAD AND NECK SIZE AND MEASUREMENT INFORMATION

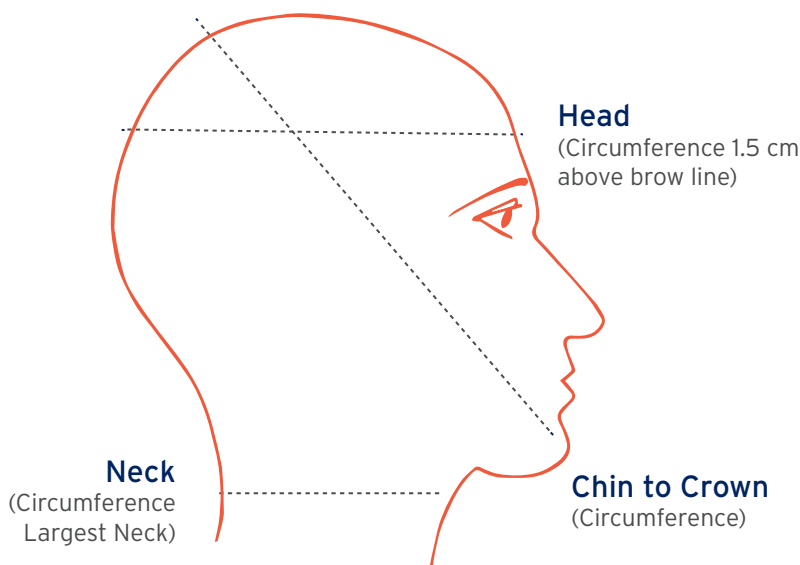

CIRCUMFERENCE IN CENTIMETERS			
Size	Head	Neck	Chin to Crown
<b>Small</b>	53.3 - 55.9	28 - 33	30 - 63.5
<b>Medium</b>	55.9 - 58.4	33 - 38.1	63.5 - 67.5
<b>Large</b>	58.4 - 61	38.1 - 43.2	67.5 - 71
<b>X-Large</b>	61 - 63.5	43.2 - 48.3	71 - 75
<b>2X-Large</b>	63.5 - 66	48.3 - 52.1	75 - 78.5

ADDITIONAL ITEMS (Sizes based on garment sizes)	
<b>Universal Neck Pad</b> for the Standard Chin Strap or Extended Chin Strap	S/M L/XL
<b>Peri-Auricular Neck Pad</b> for the Extended Chin Strap	S/M L/XL
<b>Velcro Extender</b>	One size

Take measurements when the edema is at its most reduced state. It is recommended that you mark the locations with a non-permanent, non-toxic marker at each point.

VELCRO® is a registered trademark of Velcro® Industries, B.V.  
Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,  
an Essity brand



/JOBSTUSA



@JOBSTforUSA



@JOBST\_USA



jobst-usa.com

BSN Medical Inc., an Essity company  
5825 Carnegie Blvd., Charlotte, NC 28209-4633  
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325  
To order toll-free: Tel. (+1) 800 537 1063

63030 R3 ©2023 Essity HMS North America Inc. K23